


## Instructions for Installing Sierra Wave Language Translation Files

1. Download the appropriate language translation file (.exe) and save it to your Desktop.
2. Double-click on the file to Unzip it.
3. Click the **Unzip** button. Several files will be extracted to the C:\Cadwell\Sierra Wave folder.

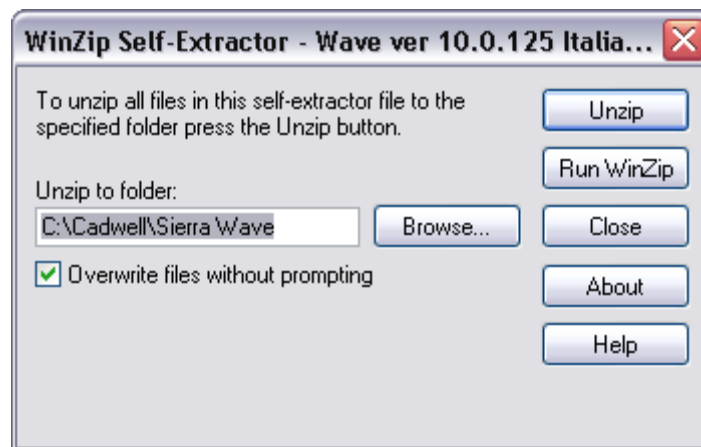

4. After the files are successfully unzipped, click the **Close** button.
5. Launch the Sierra Wave program and verify that the appropriate language is displayed.



### **Note on AnatomyView:**

In order for the nerve conduction data to be mapped correctly in the AnatomyView window you will need to update the Nerve Mapping file.

1. Launch the Sierra Wave program.
2. From the **Edit menu**, select **Test Menu**.
3. Select the **NCV** test protocol in the **Test List**.
4. Click on the **General Settings** tab.
5. Click on the **AnatomyView Nerve Map** button.
6. Click the **Clear All** button.
7. **Drag and Drop** the **nerve segments** from the right-hand column to the appropriate **Right and Left Mapped ID columns**.
8. Click **OK**.