


Instructions for Installing the Windows Vista / 7 Help Topics Patch

1. Download the appropriate file for your operating system and save it to your Desktop.
For **Windows Vista** – download "**HelpTopicsPatch.exe**".
For **Windows 7** – download "**Win7_HelpTopicsPatch.exe**".
2. Double-click on the file to Unzip it.
3. Click the **Unzip** button. A new folder will be created on your desktop, call HelpTopicsFix.

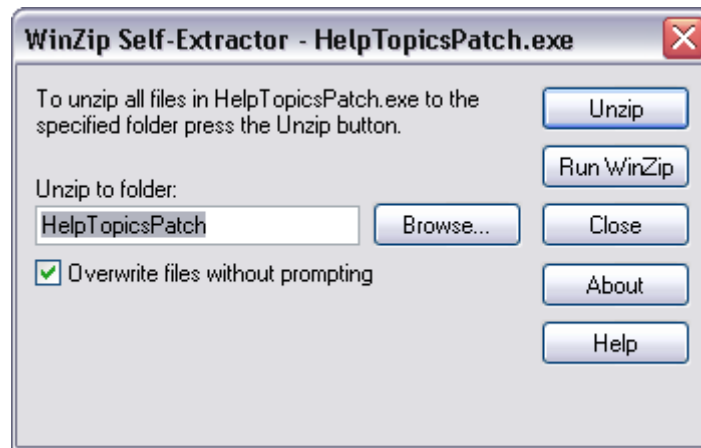

4. After the files are successfully unzipped, click the **Close** button.
5. Double-click on the **HelpTopicsPatch** folder to open it.
6. Double click on the file "**Windows6.0-KB917607-x86.msu**" for Windows Vista or on the file "**Windows6.1-KB917607-x86.msu**" for Windows 7. Follow the on-screen installation instructions.
7. When the installation is complete, double-click on the file "**WinHelp.reg**". Answer "Yes" and "OK" to the messages that are displayed.
8. Restart the computer.
9. You can now delete the downloaded file and HelpTopicsPatch folder from your desktop.