


## Instructions for Installing the ReportGen patch for Windows Vista / 7

1. Download the file "**ReportGenPatch.exe**" and save it to your Desktop.
2. Double-click on the file to Unzip it.
3. Click the **Unzip** button. Several files will be extracted to the C:\Cadwell\QuickMed folder.

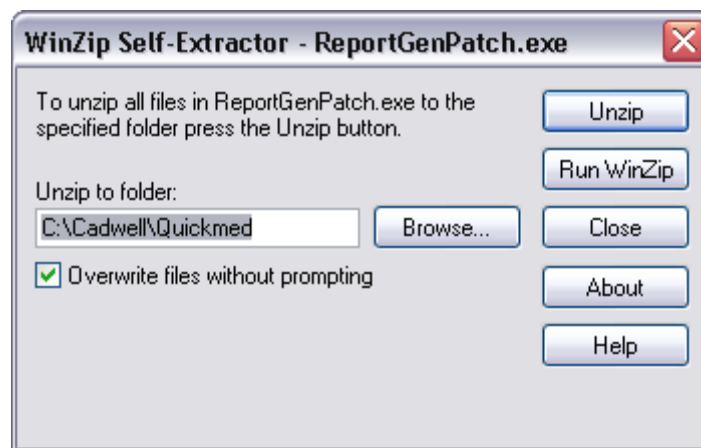

4. After the files are successfully unzipped, click the **Close** button.
5. Go to My Computer.
6. Double-click on the C:\ drive.
7. Double-click on the Cadwell folder.
8. Double-click on the QuickMed folder.
9. Find the file "**RegisterCWCOM.bat**" and double-click on it.
10. Restart the computer.