

Install Instructions

Sierra XP 3.3.3 Website Download

The download is a single self-extracting zip file that has been password protected. Double-click on the downloaded file to start the extraction process.

After entering the password the file will be unzipped and a folder called "Sierra 3.3.3" will be created in the same location as the downloaded file. Inside this folder is the normal "Setup.exe" file which can be launched to start the Sierra XP software installation.

Also included is a DSP and Help folder. These folders contain the most recent version of the base unit firmware (1.94) and Help Topics.

To Install the Sierra XP Software:

1. Double-click on the **Setup.exe** file. The Install Utility window will be displayed.
2. Click on the **Install Sierra XP Version 3.3.3** button and follow the on-screen instructions.

To Update the Base Unit Firmware:

1. Open the DSP folder.
2. Double-click on the file "**CopyFiles.bat**". This will copy the new firmware files to the C:\Cadwell\Sierrawin folder on the hard disk drive.
3. Open My Computer and go to the C:\Cadwell\Sierrawin folder.
4. Double-click on the file called "**UpdateController**". This will install the new firmware in the base unit.
5. After the install is complete, turn **OFF** the base unit for a few seconds, then turn the base unit back **ON**.
6. Start the Sierra XP program and verify the base version number by select About Sierrawin from the Help menu. The version number should be **1.94**.

To Update the Help Topics:

1. Open the Help folder.
2. Double-click on the file "**UpdateHelp.bat**". This will copy the updated help topics to the C:\Cadwell\Sierrawin folder on the hard disk drive.