

SIERRA XP VERSION 3.3.3

Purpose of the Sierra XP 3.3.3 Upgrade


- Fix problem with Store key in NCV test protocol. This fixes the problem of missing NCV tables in the TabData window.
- Fix problem with NCV cursor values when using Spanish (Mexican) for the regional settings.

**Note: French language is not supported with this version.


Installation Instructions:

Follow these steps exactly. If there is a step that you don't understand please call Cadwell Laboratories for instructions. The steps must be completed in the exact order shown below. **Phone:** (800) 245-3001 or (509) 735-6481


1. Turn On the Sierra base unit (make sure the 10 Base T network cable is connected between the Laptop or PC and the base). Turn On the Laptop or PC computer. Allow the computer to boot to the Windows desktop. Make sure the Amplifier is connected to the base.
2. Insert the Sierra XP Version 3.3.3 CD. The **SierraXP Install Utility** window should automatically be displayed. If the menu is not displayed, run the **Setup.exe** program on the CD (*Double click My Computer, then double-click on the icon for your CD-ROM drive. Then, double-click on the Setup.exe file*).
3. Click on the **Install Sierra XP Version 3.3.3** icon. Follow the on-screen instructions.

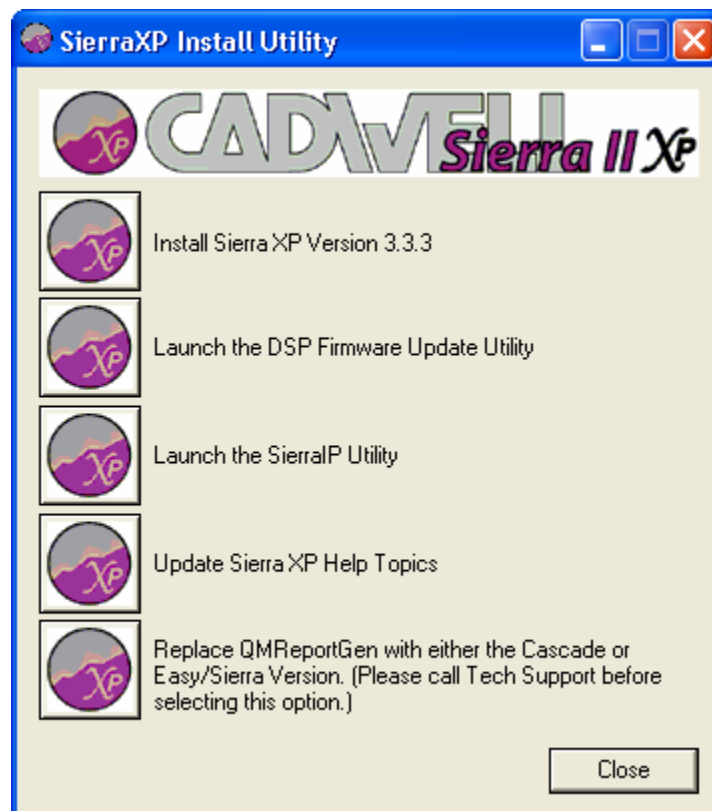

- Now you need to upgrade the firmware in the base unit. Click on the **Launch the DSP Firmware Update Utility** icon. The EthernetLoad window will be displayed. Click on the **Download** button for the Controller program (the button at the top-right of the window). This will download the latest firmware to the base unit.

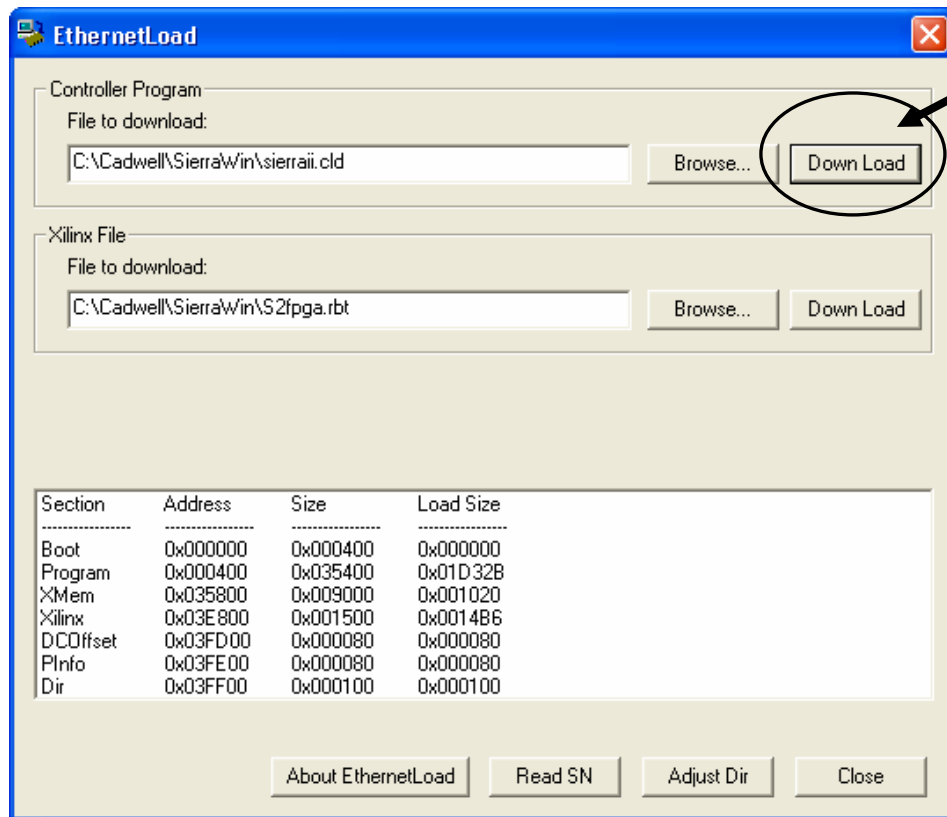

- After the firmware download is completed click the **Close** button.
- Turn OFF** the base unit and wait about **3 seconds**, then turn the **base unit back ON**.
- Now click on the **Update Sierra XP Help Topics** icon. The most recent version of the application Help Topics will be installed on the computer.
- Close the SierraXP Install Utility window and remove the CD from the drive. *Do not run any of the other installation icons unless you are instructed to do so by Cadwell personnel.*
- From the Windows Desktop, double click the **Sierra XP** icon to start the program.
- Check for the appropriate version numbers. From the **Help Menu**, select **About SierraWin**. The software version number should be **3.3.3** and the base version number should be **1.94**. *See picture on next page.*

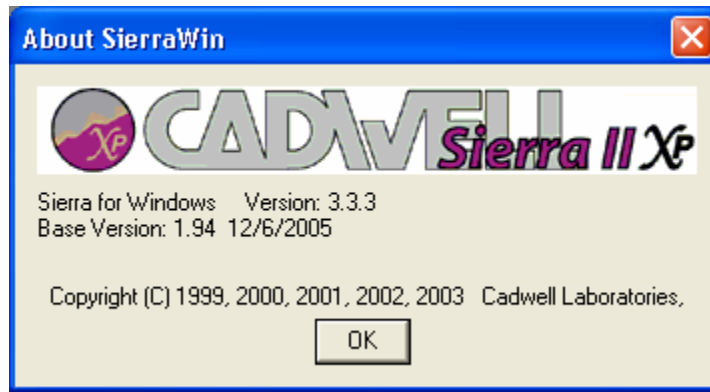

Remove the old Sierra Icon from the Desktop:

If you are upgrading from a previous version (3.2.17) of Sierra software you can delete the old Sierra icon from the Windows desktop after the installation is completed. Simply right-click on the icon and select delete.



Additional Word & PDF documents included on the CD:

- Basic Operating Instructions for the Sierra II.
- Performing Evoked Potentials on the Sierra II.
- Report Generation and Template Editor Description.
- What's New in Sierra XP 3.3.1 and 3.3.2.