



SEAMLESSLY INTRODUCE HSAT INTO YOUR PROGRAM WITH APNEATRAK

Initialize, record, score and review both HSATs and PSGs, so that all of your sleep diagnostics are performed using the same, familiar, efficient software.

CONFIDENT, COMFORTABLE HSAT

With ApneaTrak[®] HSAT devices, thoughtful design helps your patients confidently and comfortably complete their testing. You can allow your patients to manually start the study at their convenience, or predefine an automatic start time to help improve study success rates. Set up is simple with color-coded port connections, anatomical imagery, and patient instructional videos and guides.

The built-in, rechargeable battery records up to three nights on each charge and saves you the cost and headache of dealing with disposable batteries. Non-proprietary connections give you the freedom to choose the accessories you prefer.



Recessed connections are protected, and elevated from the patient's body to improve comfort and reduce overall size.

**SCHEDULE YOUR LIVE OR VIRTUAL DEMO
1-800-245-3001**



APNEATRAK CORE

Record all required channels for Type 3 testing to rule-in sleep apnea with the greatest simplicity for patients.

APNEATRAK LEGACY

Capture more respiratory detail from additional snore and thermistor channels.

APNEATRAK TST

Record three channels of ExG (EEG, EOG, EMG or ECG). Add EEG, EMG and EOG in order to calculate Total Sleep Time and improve the sensitivity of your HSAT. Two channels share a common reference; one channel is bipolar.

CORE	LEGACY	TST	CHANNELS
✓	✓	✓	Pressure Cannula
✓	✓	✓	Oximetry
✓	✓	✓	Body Position
✓	✓	✓	Thoracic Respiratory Effort
.	✓	✓	Abdominal Respiratory Effort
.	✓	✓	Thermal Airflow
.	✓	✓	Snore Sensor
.	.	✓	3 channels of ExG

SIMPLY CONNECT IT

Connecting ApneaTrak devices to your PC greets you with a window requiring minimal patient information, readying the device for the patient to take home.

After the study is complete, a single connection to the PC automatically launches the software, downloads the recording, clears, recharges, and prepares ApneaTrak to be initialized for your next patient.



APNEATRAK DISPOSABLE KITS ARE EXCLUSIVE TO APNEATRAK

No interface cables required! Save time cleaning after every patient and reduce possible infection control issues. On return, keep the device and throw everything else away! Reusable kits are also available.

Visit www.estore.cadwell.com or contact your Cadwell Representative for more information regarding accessories and consumables for ApneaTrak HSAT products.